


## ANLEITUNG / INSTRUCTION

FFP2 - FAGO S 101

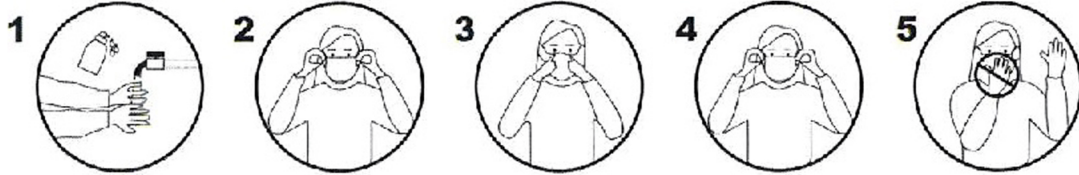

DE

### ANWEISUNGEN ZUM ANLEGEN

1. Halten Sie die Halbmaske über die Nase und den Mund.
2. Ziehen Sie die Kopfbänder hinter die Ohren, und bringen sie die Bänder am Clip an. Passen Sie den Sitz an, und überprüfen Sie die Dichtheit.
3. Passen Sie den Nasenclip so an, dass er sicher um die Nase liegt, drücken Sie den Nasenclip fester an. Wiederholen Sie die Anpassungen, bis die Maske dicht ist.
4. Berühren Sie nicht die Vorderseite der Maske beim tragen.

#### Warnhinweise

1. Diese mit dem Zeichen „NR“ versehene Maske ist zur einmaligen Verwendung bestimmt.
2. Diese Maske hilft beim Schutz vor bestimmten Partikelverunreinigungen, verhindert jedoch nicht vollständig das Risiko auf Ansteckung oder Infektion.
3. Sofern Gesichtshaar unter die Gesichtsversiegelung gerät, werden die Dichtheitsanforderungen ggf. Nicht erfüllt.
4. Wechseln Sie umgehend die Maske, wenn Ihnen das Atmen schwerfällt oder die Maske beschädigt bzw. deformiert wird.
5. Wechseln Sie die Maske, wenn die Dichtheit über dem Gesicht nicht erreicht werden kann.

#### Lagerung

Lagern Sie die Maske bei Nichtverwendung in einem dichten Behälter außerhalb der verunreinigten Bereiche.

#### Schutzschutufe

FFP2 NR mit Schutzgrad 95%

EN

### FITTING INSTRUCTIONS

1. Hold the half mask over your nose and mouth.
2. Pull the headbands behind your ears and attach the straps to the clip. Adjust the seat and check for leaks.
3. Adjust the nose clip so that it fits securely around your nose, then press the nose clip tighter. Repeat the adjustments until the mask is tight.
4. Do not touch the front of the mask while wearing it.

#### Warning

1. This mask with the "NR" mark is for single use only.
2. This mask helps protect against certain particulate contaminants, but does not completely prevent the risk of contagion or infection.
3. If facial hair gets under the face seal, the tightness requirements may not be met.
4. Change the mask immediately if you cannot breathe or if the mask is damaged or deformed.
5. Change the mask if the tightness over the face cannot be achieved.

#### Storage

When not in use, store the mask in a tight container away from the contaminated areas.

#### Protection level

FFP2 NR with degree of protection 95%